

EARWELL®

WEAR & CARE GUIDE



ADHESIVE CARE:

- EarWell is held in place with a medical-grade adhesive that is secure, yet gentle on your baby's skin.
- It will typically stay in place for 14–17 days. Please check it daily to ensure it is fully attached.
- **If the adhesive begins to lift:**
 - Call your clinic to schedule an appointment as soon as possible.
 - While waiting, secure it with the provided tape: cut a small piece (about 3 inches long), place it over the top of the EarWell and onto skin. Use more tape to secure it if needed.
- If more than 50% of the EarWell cover lifts or it comes completely off. **Do not reattach it. Call your clinic immediately.**
- A brief dislodgement won't affect results, but prompt reattachment is important.



KEEPING IT DRY:

- Keep EarWell dry—the adhesive works best when it is not wet.
- Do not immerse the EarWell in water or wash it.
- Use a damp washcloth to clean around the EarWell and scalp.
- Avoid baths or activities that may wet the EarWell.



WATCH FOR REDNESS

- EarWell is clear so you can see your baby's ear.
- **If you notice unusual redness around their ear, please contact your clinic.**



KEEPING IT CLEAN

- Keep the inside of the EarWell free of milk, formula, or spit-up.
- Place a washcloth under your baby's cheek during feedings to prevent spills.
- If fluid enters the EarWell, call your clinic. They may ask you to come in for a cleaning and a component check.



COVER REMOVAL

- **Do not attempt to remove or adjust the EarWell yourself.**
- **Only your clinician should remove the top cover.**

If you have any questions at any point during your infant's EarWell Therapy, please contact your local Cranial Technologies® clinic. We're here to support you every step of the way.

EarWell®
INFANT EAR CORRECTION SYSTEM

| from |

Cranial
Technologies®