

EARWELL®

WEAR & CARE GUIDE



ADHESIVE CARE:

- EarWell is held in place with a medical-grade adhesive that is secure, yet gentle on your baby's skin.
- It will typically stay in place for 14–17 days. Please check it daily to ensure it is fully attached.
- **If the adhesive begins to lift:**
 - Call your clinic to schedule an appointment as soon as possible.
 - While waiting, secure it with the provided tape: cut a small piece (about 3 inches long), place it over the top of the EarWell and onto skin. Use more tape to secure it if needed.
- If more than 50% of the EarWell cover lifts or it comes completely off. **Do not reattach it. Call your clinic immediately.**
- A brief dislodgement won't affect results, but prompt reattachment is important.



KEEPING IT DRY:

- Keep EarWell dry—the adhesive works best when it is not wet.
- Do not immerse the EarWell in water or wash it.
- Use a damp washcloth to clean around the EarWell and scalp.
- Avoid baths or activities that may wet the EarWell.



WATCH FOR REDNESS

- EarWell is clear so you can see your baby's ear.
- **If you notice unusual redness around their ear, please contact your clinic.**



KEEPING IT CLEAN

- Keep the inside of the EarWell free of milk, formula, or spit-up.
- Place a washcloth under your baby's cheek during feedings to prevent spills.
- If fluid enters the EarWell, call your clinic. They may ask you to come in for a cleaning and a component check.



REMOVAL

- Do not attempt to adjust the EarWell device yourself or remove the top cover unless complete removal of device is necessary.
- Follow directions for removal as instructed by your clinician and provided on the EarWell Removal handout.

If you have any questions at any point during your infant's EarWell Therapy, please contact your local Cranial Technologies® clinic. We're here to support you every step of the way.

EarWell®
INFANT EAR CORRECTION SYSTEM

| from |

Cranial Technologies®

INS042 Rev00, ECO26-058, Effective Date: 03/16/2026